

X-PANDER TRAINING BAR

Instructions for use in training



The tongues serve only for positioning and are not involved in the operation of the tourniquet.

It is important that they are facing upwards!



The blue **X-Pander Training Bar** is used with a tourniquet from which the original bar must be removed, appearing as shown in the photo.



Take the blue **X-Pander Training Bar** and insert the straps into the slot, slightly raising the tongue.



Push the 2 straps towards the opposite end of the slot and...



... help the insertion of the 2 straps into the slot by crushing them in the opposite end of the slot.



Extend the 2 straps inside the slot pushing them until they are under the short tongue.



X-PANDER TRAINING BAR

Universal bar for tourniquet training

The blue-coloured X-Pander Training Bar can be used on all expired “CAT® type” tourniquets and those with original operational bars damaged to the extent that the tourniquet is no longer safe, or whenever the operation of a tourniquet is compromised and the out-of-use device is assigned to training.

Before fitting, it is important to remove the old bar from the out-of-use tourniquet.

Fitting the X-Pander Training Bar to an out-of-use tourniquet transforms it into a training device, easily identified by the blue colour and unusual design. It can then be used multiple times, providing enhanced safety and cost savings.



Via Biandrate, 80
28060 SAN PIETRO MOSEZZO (NO) - ITALY
Tel. +39 0321 442441 - Fax +39 0321 392190
email: info@flamor.com

